



DRAWING DIMENSIONS REFER TO CYROLITE MATERIAL

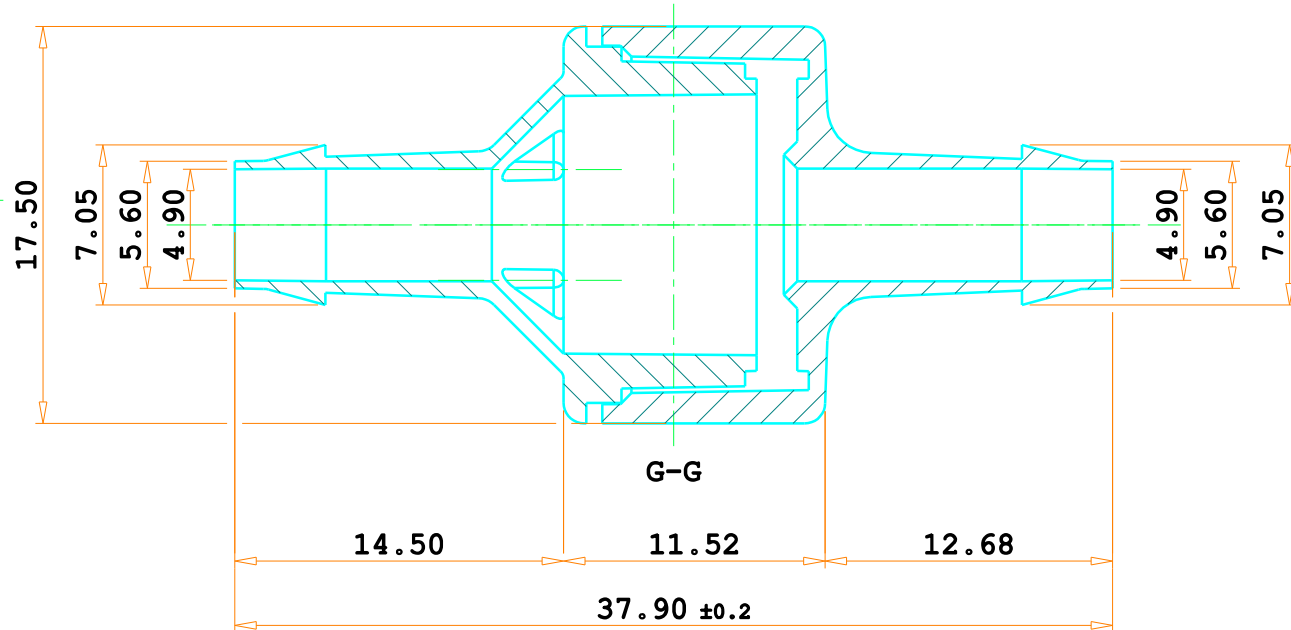

Product code	Moulding material
ST341B1-4C	CYROLITE G20
ST341B1-4T	PC TRIREX 3022
ST341B1-4Z	ZYLAR 550

 **CARRO LUIGI Srl**
25030 RUDIANO (BS)
Via Lavoro & Industria


M0803.04 Rev.00

General tolerances ISO 2768-mk	
0.5 < x < 6	+0.1
6 < x < 30	+0.2
30 < x < 120	+0.3
120 < x < 400	+0.5

Mould Denomination
HIGH FLOW CHECK VALVE

Member Denomination

HIGH FLOW CHECK VALVE WITH BARBED 1/4 (A23)

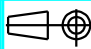Date	Molding material	C.tion	Machine	Mould weight	
21/09/2022	See Table				
Drawn by	Approved by		Eyebolts	Mould dimensions	
M. Cassani	Direzione C.A.				
N°Par.	N°Pez.	Dimensions	Material	Treatment	Scale
	2				

N°Cavities
4

Order
M1999


Customer code
4010022

Customer part
ST341B1-4X

 Sheet
06

3	2	1	Changes
			Signature
			Date

DRAWING DIMENSIONS REFER TO CYROLITE MATERIAL

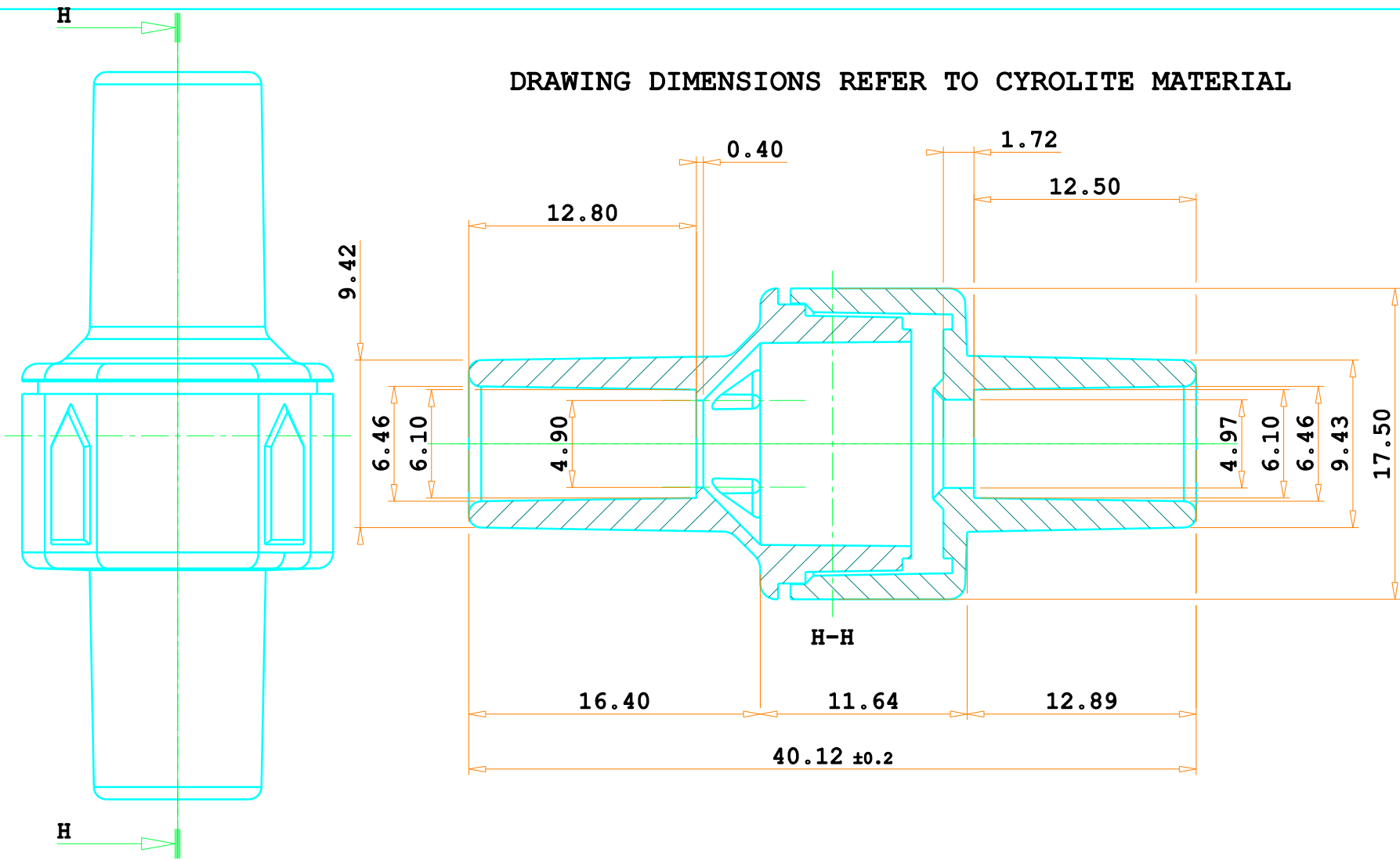

Product code	Moulding material
ST3411-4C	CYROLITE G20
ST3411-4T	PC TRIREX 3022
ST3411-4Z	ZYLAR 550

CARRO LUIGI Srl
 25030 RUDIANO (BS)
 Via Lavoro & Industria

M0803.04 Rev.00

General tolerances		ISO 2768-mk
0.5	< x < 6	+0.1
6	< x < 30	+0.2
30	< x < 120	+0.3
120	< x < 400	+0.5

Mould Denomination
 HIGH FLOW CHECK VALVE

N° Cavities
 4

Order
 M1999

Member Denomination
 HIGH FLOW CHECK VALVE TUBE 1/4

Customer code
 4010022


Date	Moulding material	C.tion	Machine	Mould weight
21/09/2022	See Table			
Drawn by	Approved by		Eyebolts	Mould dimensions
M. Cassani	Direzzone C.A.			
N°Par.	N°Pez.	Material	Treatment	Scale
	2			

Customer part
 ST3411-4X

A4 Sheet
 06

3	2	1	Changes
			Signature
			Date

DRAWING DIMENSIONS REFER TO CYROLITE MATERIAL

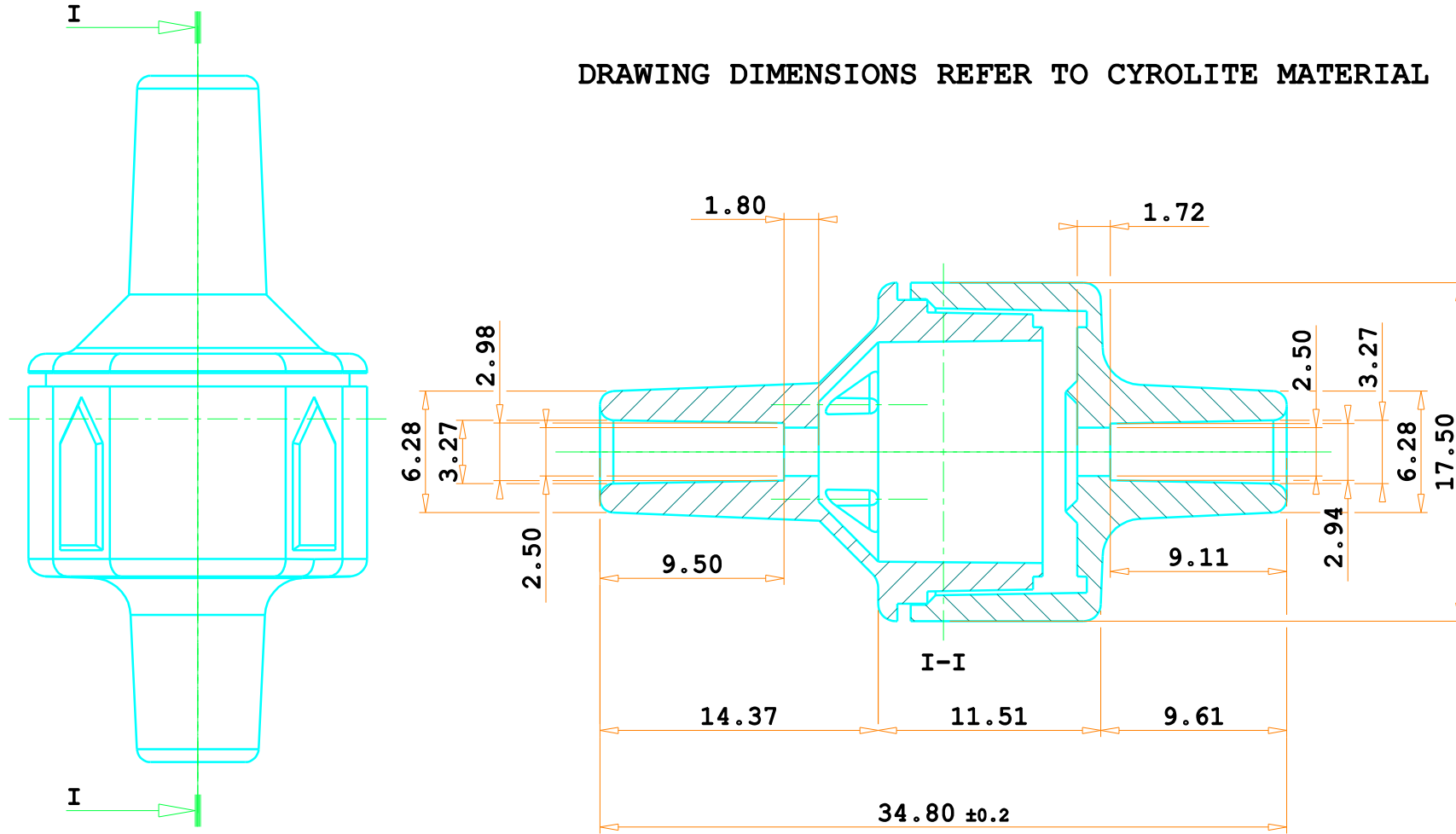

Product code	Moulding material
ST3411-8C	CYROLITE G20
ST3411-8T	PC TRIREX 3022
ST3411-8Z	ZYLAR 550

CARRO LUIGI Srl
 25030 RUDIANO (BS)
 Via Lavoro & Industria

M0803.04 Rev.00

General tolerances ISO 2768-mk

0.5 < x < 6	+0.1	.
6 < x < 30	+0.2	.
30 < x < 120	+0.3	.
120 < x < 400	+0.5	.

Mould Denomination
 HIGH FLOW CHECK VALVE

Member Denomination
 HIGH FLOW CHECK VALVE TUBE 1/8

Date
 21/09/2022

Drawn by
 M. Cassani

N°Par.
 2

Moulding material
 See Table

Approved by
 Direzione C.A.

Dimensions

C.tion



Material

Machine

Eyebolts

Treatment

Mould weight

Mould dimensions

Scale

N° Cavities
 4

Order
 M1999

Customer code
 4010022



Customer part
 ST3411-8X

A4

Sheet
 06

3
2
1

Date Signature Changes

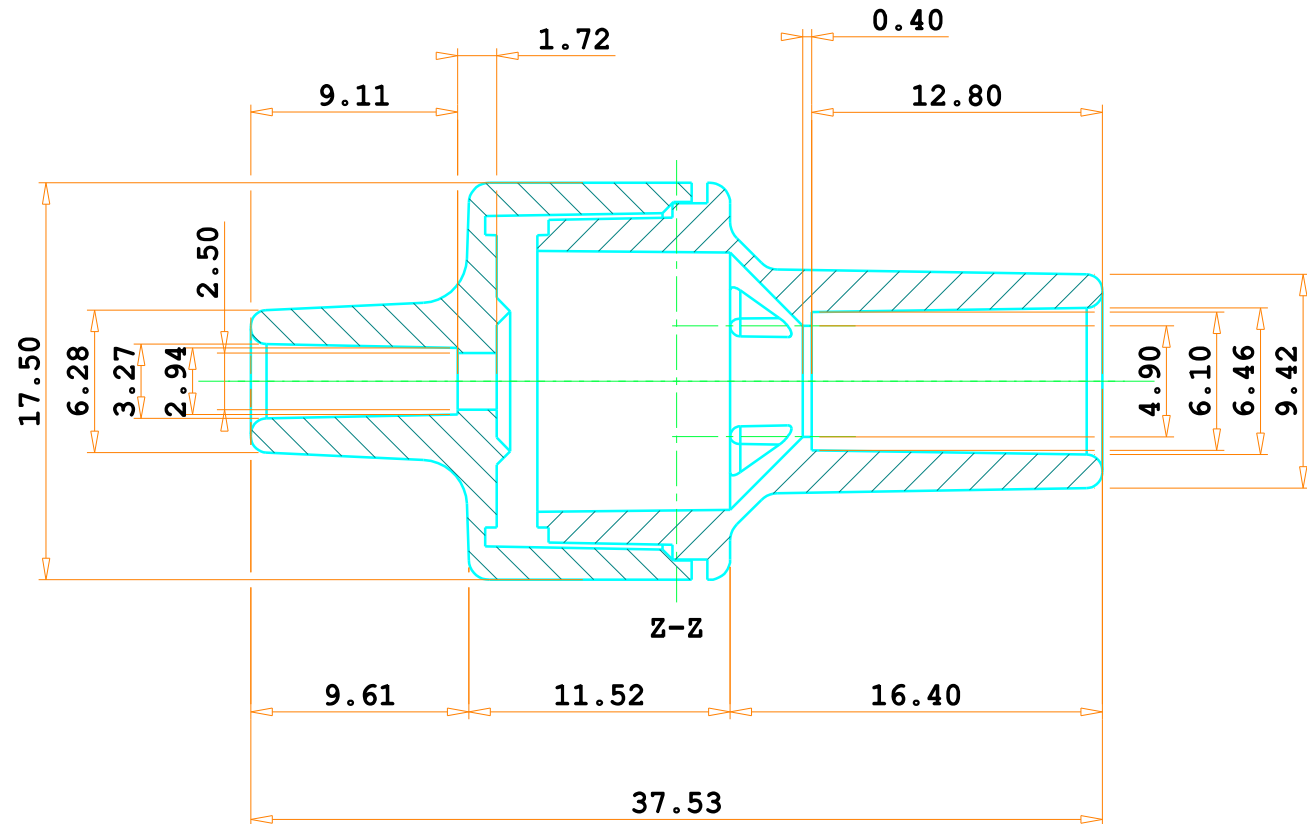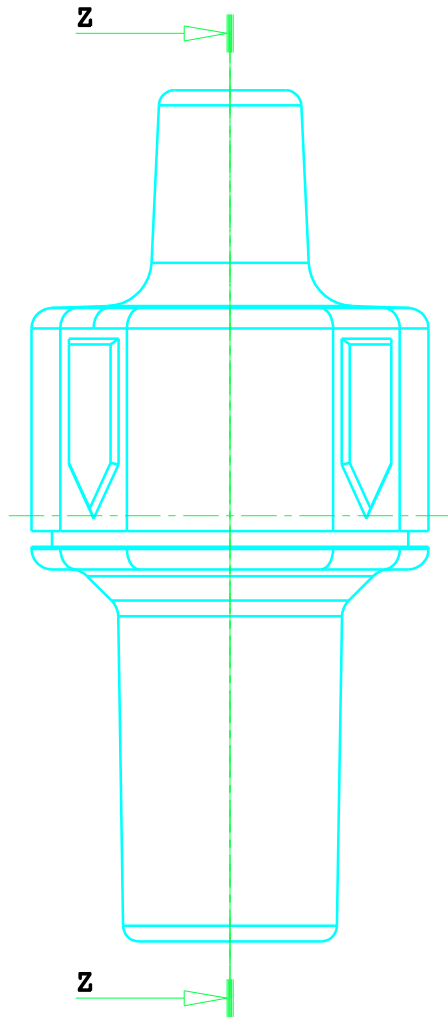

PRODUCT CODE	MOULDING MATERIAL
ST34118-14Z	ZYLAR 550
ST34118-14T	PC TRIREX 3022
ST34118-14C	CYROLITE G20

CARRO LUIGI Srl
 25030 RUDIANO (BS)
 Via Lavoro & Industria

M0703.04 Rev.01

General tolerances ISO 2768-mk

0.5 < x < 6	+0.1	-
6 < x < 30	+0.2	-
30 < x < 120	+0.3	-
120 < x < 400	+0.5	-

Mould Denomination

HIGH FLOW CHECK VALVE

Member Denomination

HIGH FLOW CHECK VALVE TUBE 1/8 TO 1/4

Date 21/09/2022	Molding material SEE TABLE	C.tion 0.6%	Machine	Mould weight
Drawn by Cassani M.	Approved by Direzione C.A.	Eyebolts	Mould dimensions	
N°Par. -	N°Pez. 1+1	Dimensions	Material	Treatment
			Scale 1:3	

N°Cavities

Order

M1999

Customer code

4010022

Customer part

ST34118-14X

A4



Sheet
1

3
2
1

Changes

Signature

Date

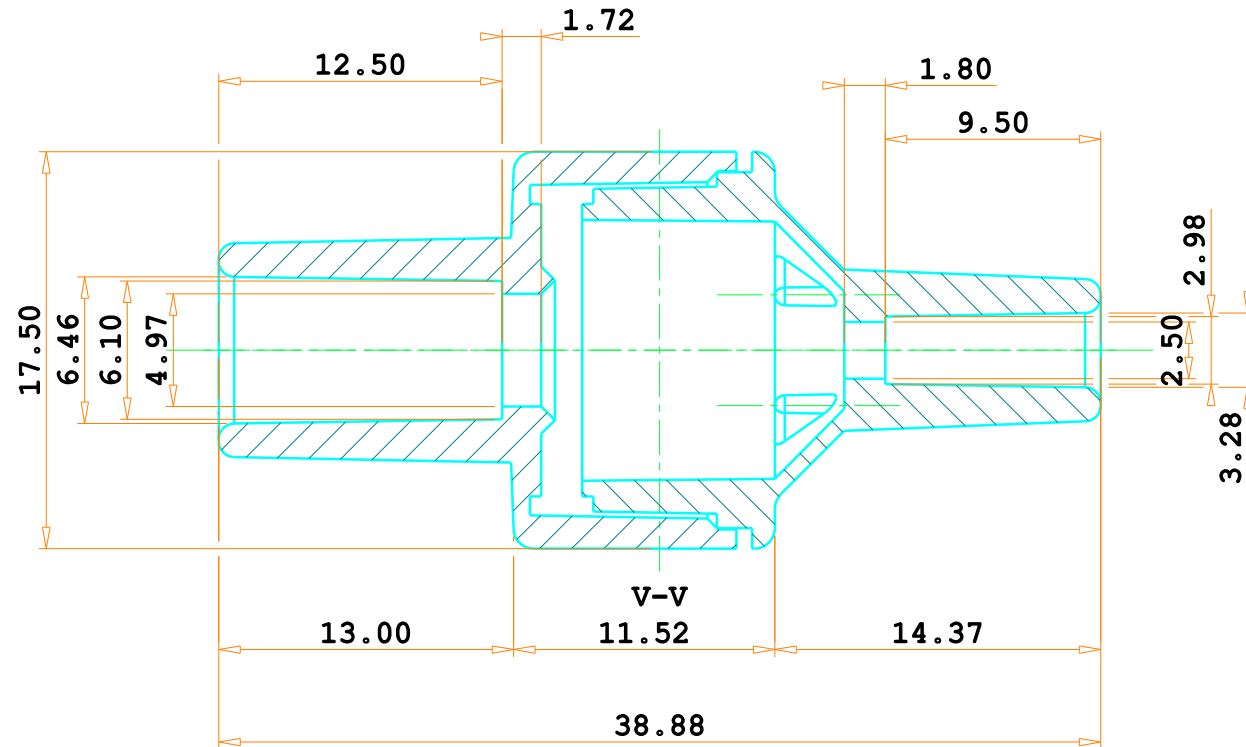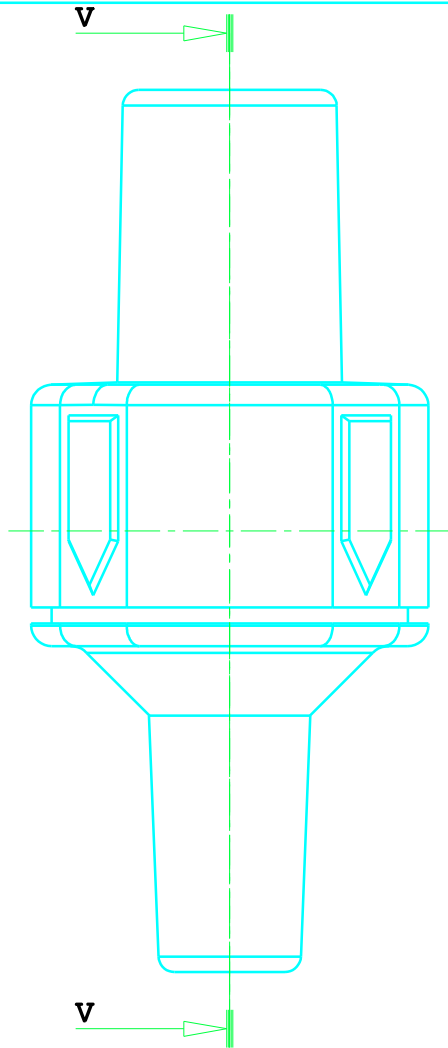

PRODUCT CODE	MOULDING MATERIAL
ST34114-18Z	ZYLAR 550
ST34114-18T	PC TRIREX 3022
ST34114-18C	CYROLITE G20

CARRO LUIGI Srl
 25030 RUDIANO (BS)
 Via Lavoro & Industria

M0703.04 Rev.01

General tolerances	ISO 2768-mk
0.5 < x < 6	+0.1
6 < x < 30	+0.2
30 < x < 120	+0.3
120 < x < 400	+0.5

Mould Denomination

HIGH FLOW CHECK VALVE

Member Denomination

HIGH FLOW CHECK VALVE TUBE 1/4 TO 1/8

Date	Molding material	C.tion	Machine	Mould weight
21/09/2022	SEE TABLE	0.6%		
Drawn by	Approved by	Eyebolts	Mould dimensions	
Cassani M.	Direzione C.A.			
N°Par.	N°Pez.	Dimensions	Material	Treatment
-	1+1			
				Scale
				1:3

N°Cavities

Order

M1999

Customer code

4010022

Customer part

ST34114-18X

A4

Sheet
1

3



2

1

Changes

Signature

Date

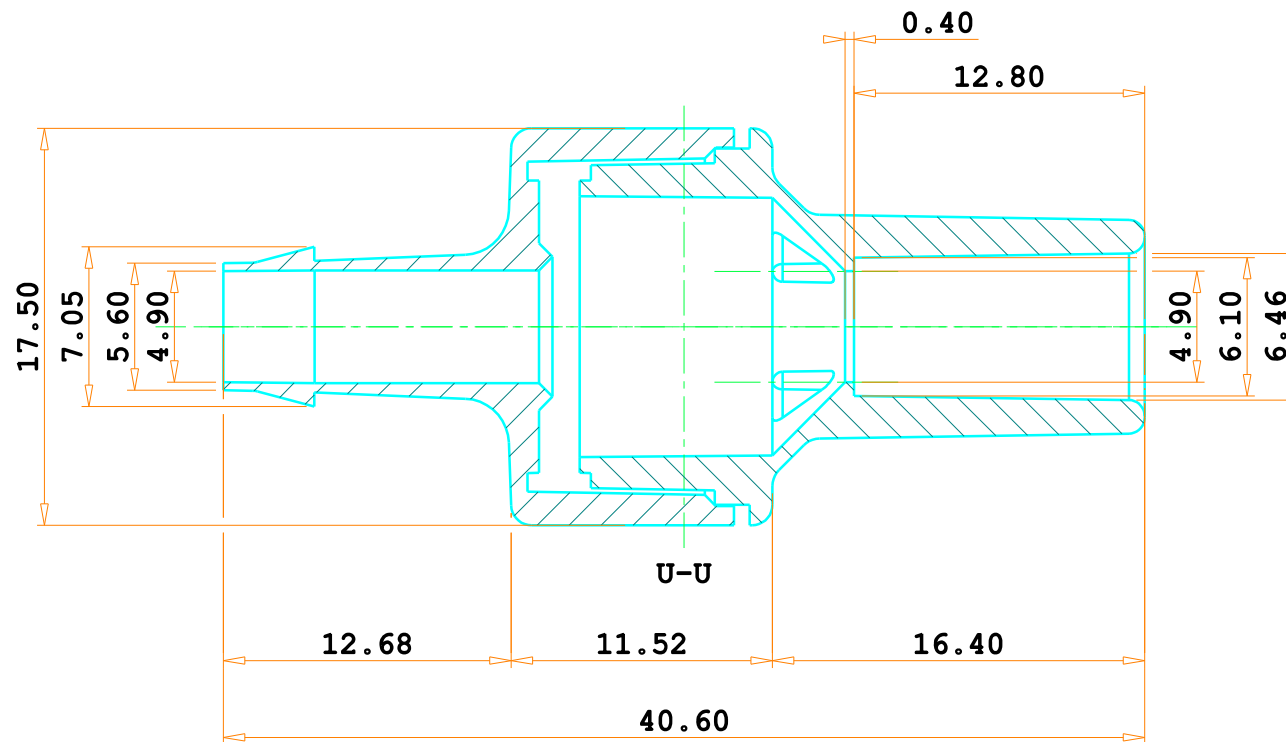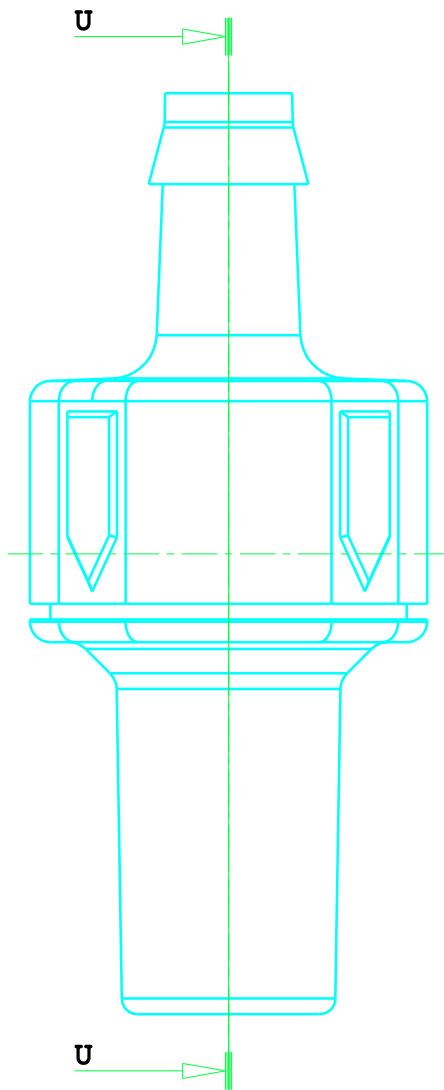

PRODUCT CODE	MOULDING MATERIAL
ST341B14-14Z	ZYLAR 550
ST341B14-14T	PC TRIREX 3022
ST341B14-14C	CYROLITE G20

CARRO LUIGI Srl
 25030 RUDIANO (BS)
 Via Lavoro & Industria

M0703.04 Rev.01

General tolerances ISO 2768-mk

0.5 < x < 6	+0.1	.
6 < x < 30	+0.2	.
30 < x < 120	+0.3	.
120 < x < 400	+0.5	.

Mould Denomination

HIGH FLOW CHECK VALVE

N° Cavities

Order

M1999

Member Denomination

HIGH FLOW CHECK VALVE BARBED/TUBE 1/4 TO 1/4

Customer code

4010022

Date

21/09/2022

Molding material

SEE TABLE

C.tion

0.6%

Machine

Mould weight

Customer part

ST341B14-14X

Drawn by

Cassani M.

Approved by

Direzione C.A.

Eyebolts

Mould dimensions

N°Par.

N°Pez.

1+1

Dimensions

Material

Treatment

Scale

1:3

A4

Sheet
1

3



2

1

Changes

Signature

Date

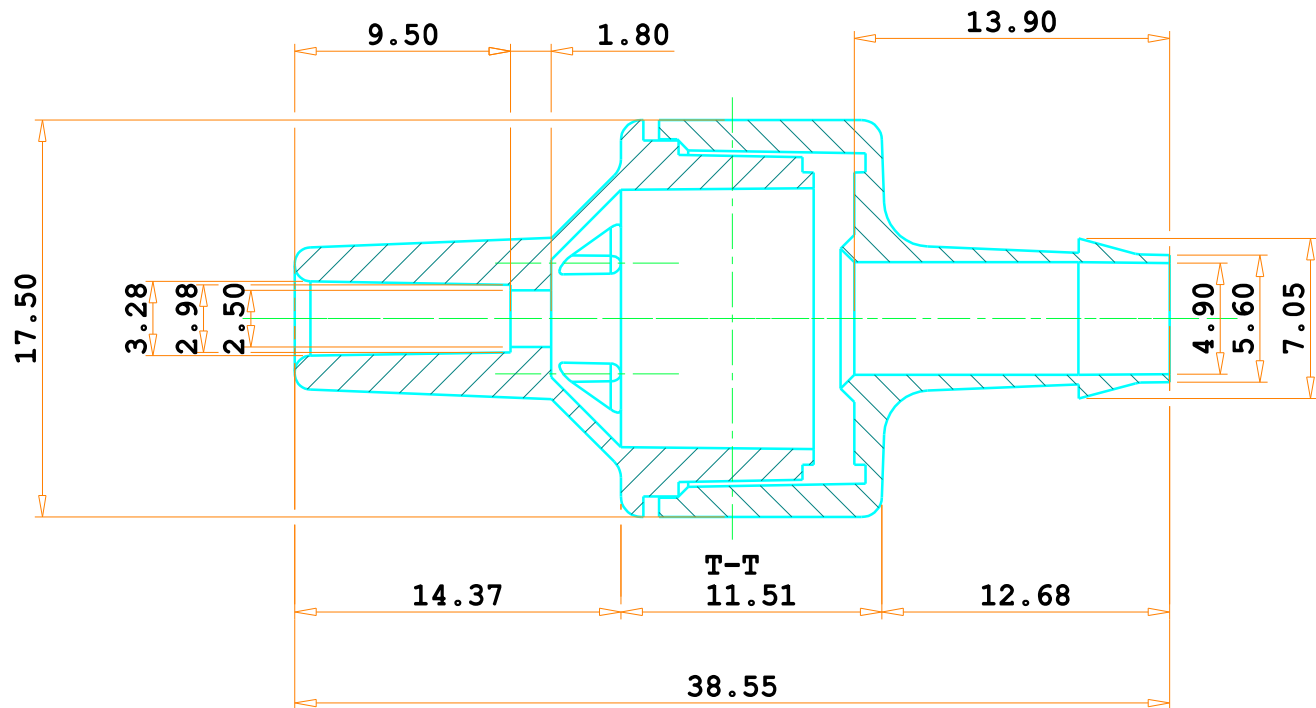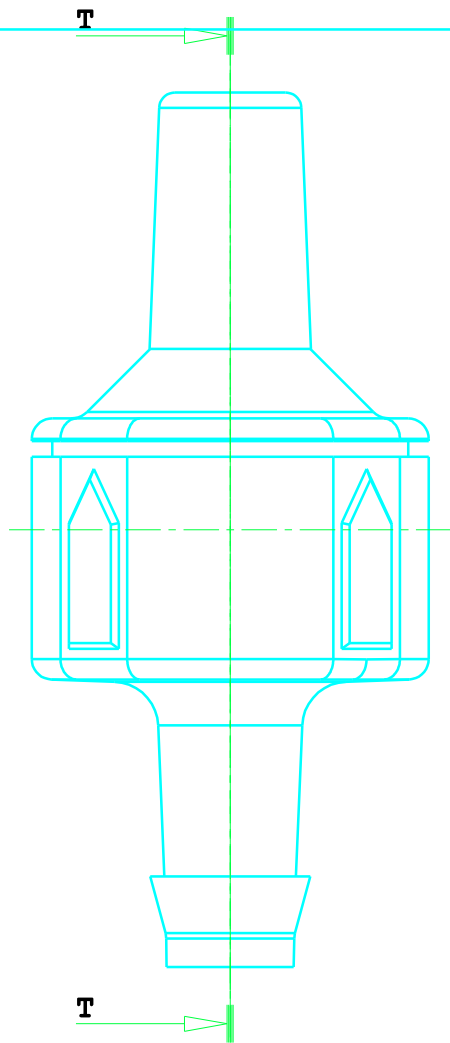

PRODUCT CODE	MOULDING MATERIAL
ST341B14-18Z	ZYLAR 550
ST341B14-18T	PC TRIEX 3022
ST341B14-18C	CYROLITE G20

CARRO LUIGI Srl
 25030 RUDIANO (BS)
 Via Lavoro & Industria

M0703.04 Rev.01

General tolerances	ISO 2768-mk
0.5 < x < 6	+0.1
6 < x < 30	+0.2
30 < x < 120	+0.3
120 < x < 400	+0.5

Mould Denomination

HIGH FLOW CHECK VALVE

Member Denomination

HIGH FLOW CHECK VALVE BARBED/TUBE 1/4 TO 1/8

Date	Molding material	C.tion	Machine	Mould weight
21/09/2022	SEE TABLE	0.6%		
Drawn by	Approved by	Eyebolts	Mould dimensions	
Cassani M.	Direzione C.A.			
N°Par.	N°Pez.	Dimensions	Material	Treatment
-	1+1			
			Scale	
			1:3	

N°Cavities

Order

M1999

Customer code

4010022

Customer part

ST341B14-18X



A4

Sheet
1

3
2
1

Date
Signature

Changes

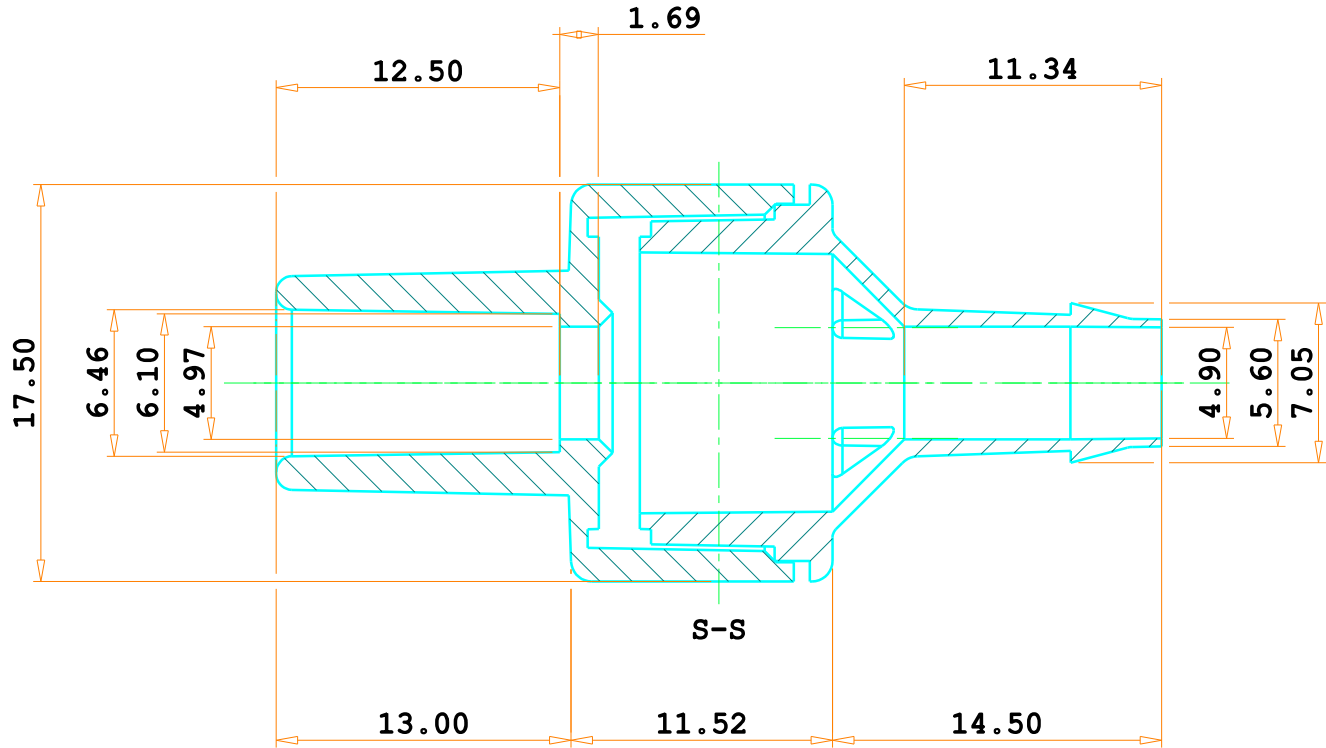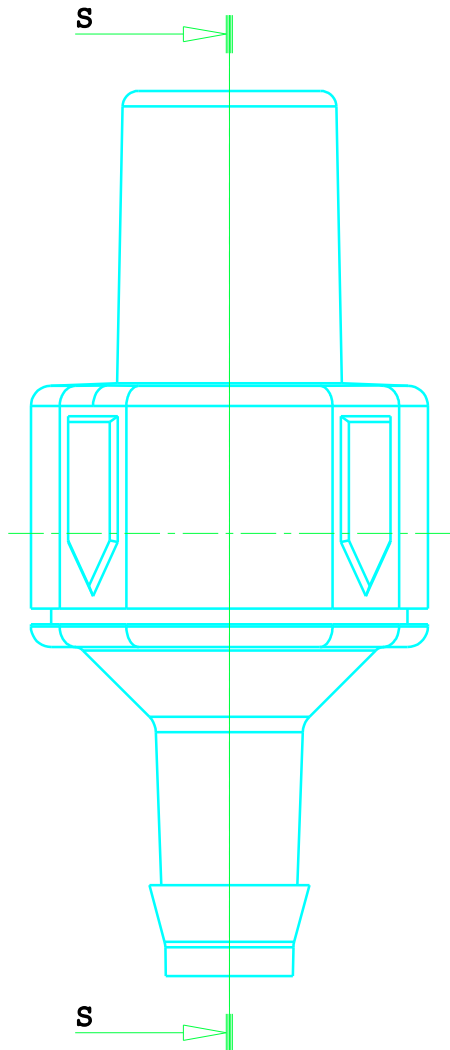

Date	Signature	1	Changes
		2	
		3	

PRODUCT CODE	MOULDING MATERIAL
ST34114-14BZ	ZYLAR 550
ST34114-14BT	PC TRIREX 3022
ST34114-14BC	CYROLITE G20

 **CARRO LUIGI Srl**
 25030 RUDIANO (BS)
 Via Lavoro & Industria

M0703.04 Rev.01

General tolerances	ISO 2768-mk
0.5 <x < 6	+0.1
6 <x < 30	+0.2
30 <x < 120	+0.3
120 <x < 400	+0.5

Mould Denomination
 HIGH FLOW CHECK VALVE

N°Cavities



Order
 M1999

Member Denomination
 HIGH FLOW CHECK VALVE TUBE BARBED 1/4 TO 1/4


Customer code
 4010022

Date	Molding material	C.tion	Machine	Mould weight
21/09/2022	SEE TABLE	0.6%	---	---
Drawn by	Approved by	Eyebolts		
Cassani M.	Direzione C.A.	---		
N°Par.	N°Pez.	Material	Treatment	Scale
---	1+1	---	---	1:3

Customer part
ST34114-14BX
Sheet
1

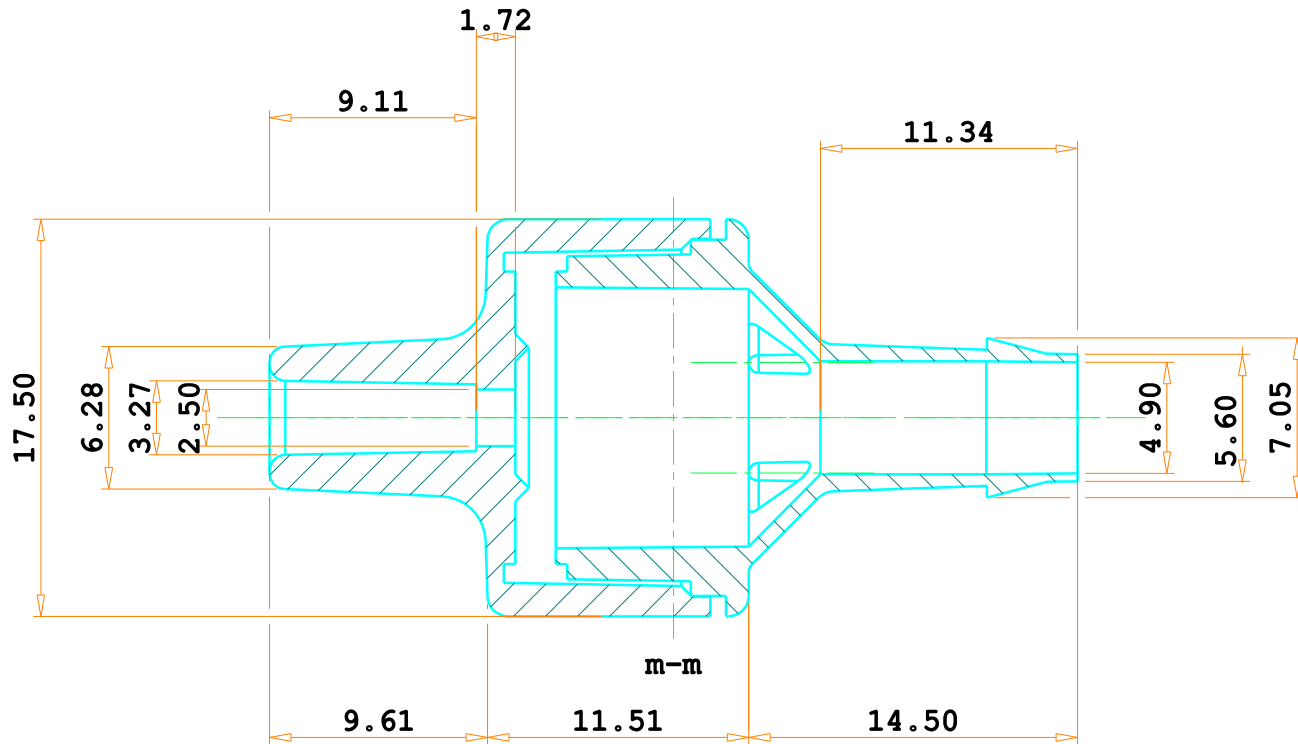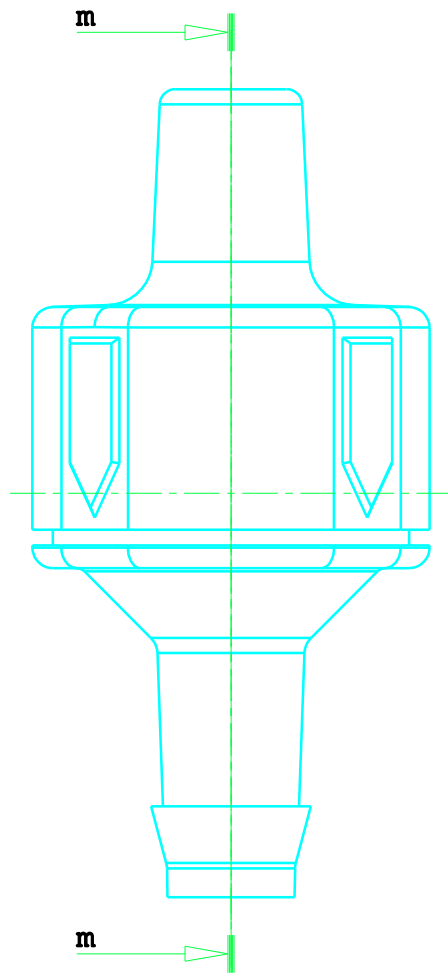

PRODUCT CODE	MOULDING MATERIAL
ST34118-14BZ	ZYLAR 550
ST34118-14BT	PC TRIREX 3022
ST34118-14BC	CYROLITE G20

 **CARRO LUIGI Srl**
 25030 RUDIANO (BS)
 Via Lavoro & Industria

M0703.04 Rev.01

General tolerances ISO 2768-mk	
0.5 < x < 6	+0.1
6 < x < 30	+0.2
30 < x < 120	+0.3
120 < x < 400	+0.5

Mould Denomination

HIGH FLOW CHECK VALVE

N° Cavities

Order

M1999

Member Denomination

HIGH FLOW CHECK VALVE TUBE BARBED 1/8 TO 1/4

Customer code

4010022

Date

21/09/2022

Molding material

SEE TABLE

C.tion

0.6%

Machine

Mould weight

Customer part

ST34118-14BX

Drawn by

Cassani M.

Approved by

Direzione C.A.

Eyebolts



Mould dimensions

N° Par.

N° Pez.

1+1

Dimensions

Material

Treatment

Scale

1:3

A4

Sheet

1

3


2

1


Changes

Signature


Date

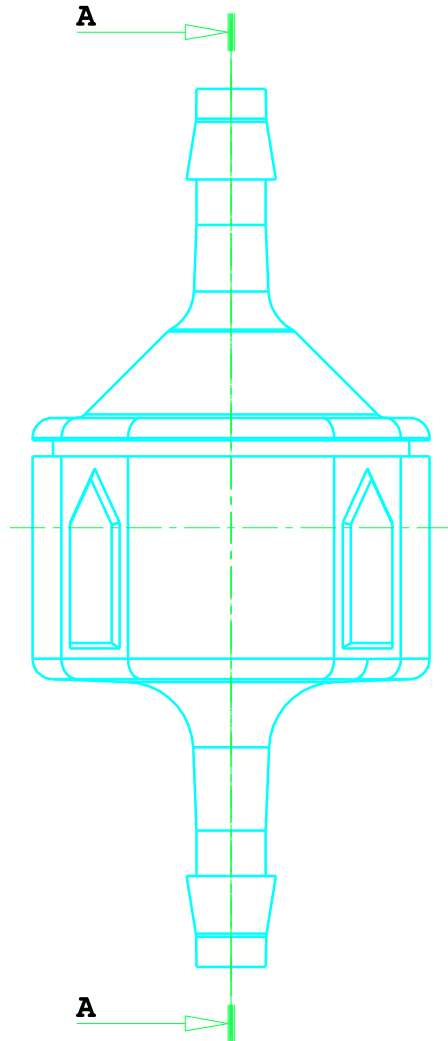

DRAWING DIMENSIONS REFER TO CYROLITE MATERIAL

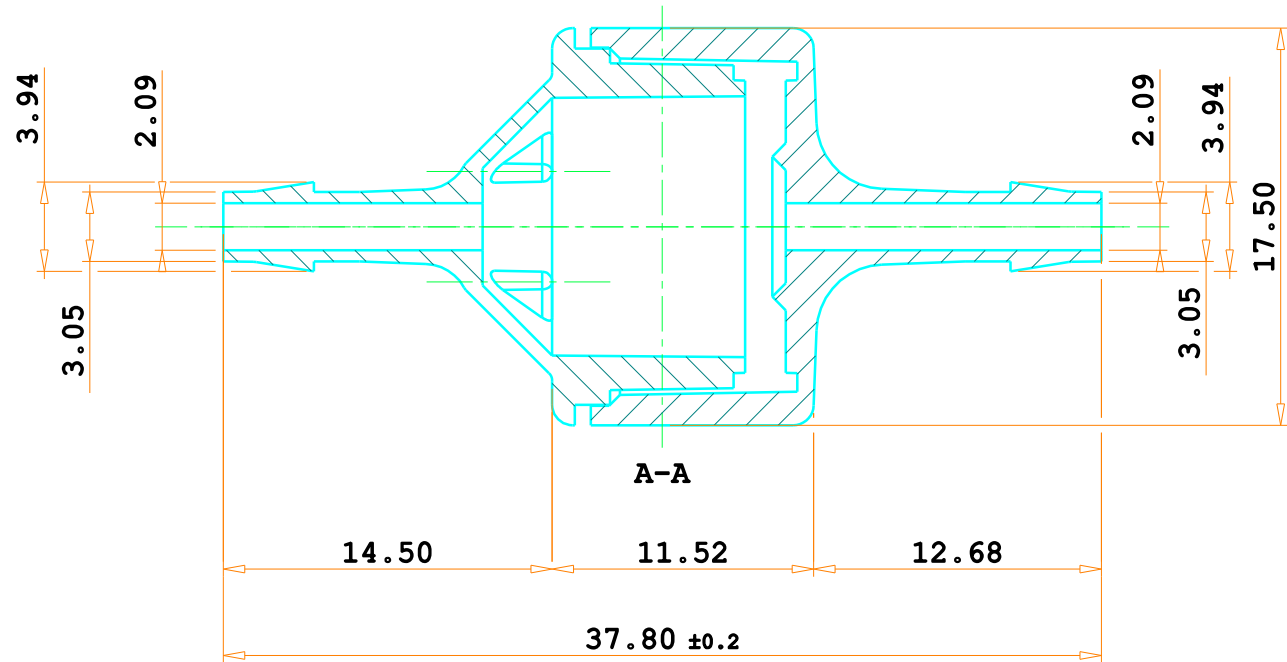

Product code	Moulding material
ST341A21C	CYROLITE
ST341A21T	TRIREX
ST341A21Z	ZYLAR

 **CARRO LUIGI Srl**
 25030 RUDIANO (BS)
 Via Lavoro & Industria

M0803.04 Rev.00

General tolerances		ISO 2768-mk	
0.5	< x < 6	+0.1	.
6	< x < 30	+0.2	.
30	< x < 120	+0.3	.
120	< x < 400	+0.5	.

Mould Denomination
 HIGH FLOW CHECK VALVE TUBE

Member Denomination
 HIGH FLOW CHECK VALVE TUBE_A-21

Date
 21/09/2022

Drawn by
 M. Cassani

N°Par.
 2

Moulding material
 See Table

Approved by
 Direzione C.A.

Dimensions

C.tion
 0.6%



Material

Machine

Eyebolts

Treatment

Mould weight

Mould dimensions

Scale
 Scale 3:1

N° Cavities
 4

Order
 M1999


Customer code
 4010022

Customer part
 ST341A21x




Sheet
 01

Date	Signature	Changes
		3
		2
		1

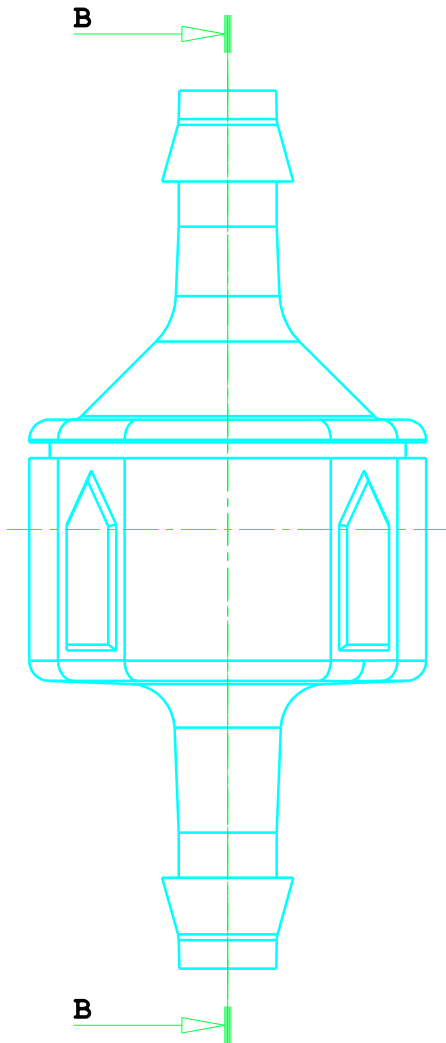

DRAWING DIMENSIONS REFER TO CYROLITE MATERIAL

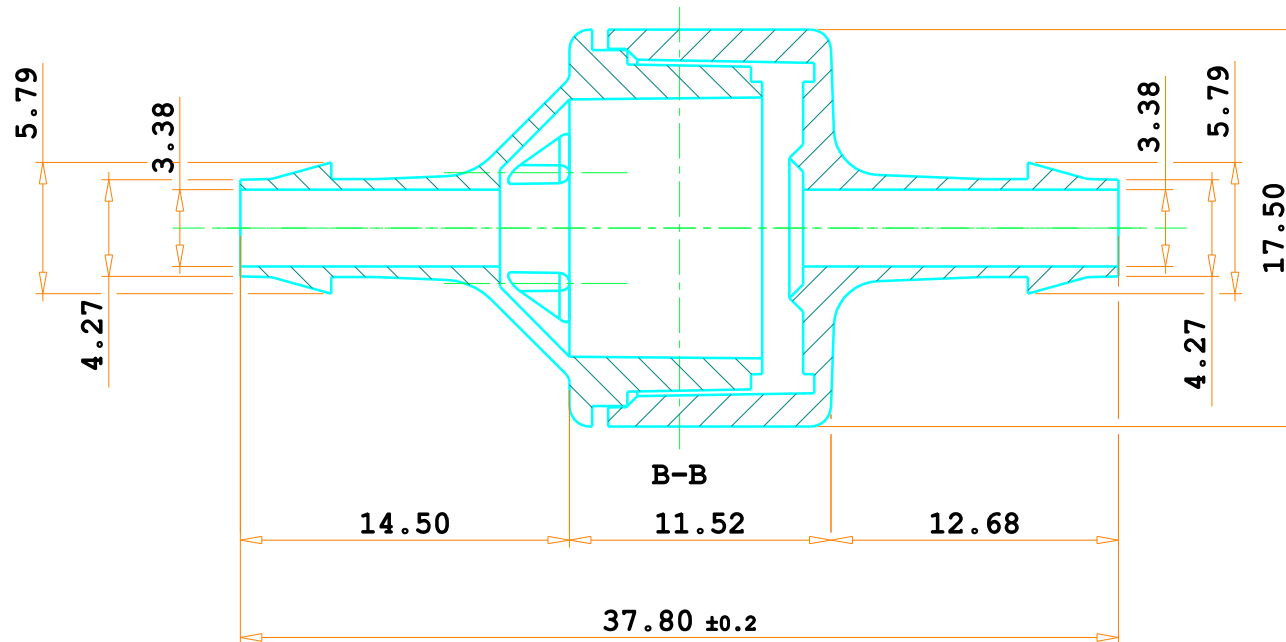

Product code	Moulding material
ST341A22C	CYROLITE
ST341A22T	TRIREX
ST341A22Z	ZYLAR

CARRO LUIGI Srl
 25030 RUDIANO (BS)
 Via Lavoro & Industria

M0803.04 Rev.00

General tolerances		ISO 2768-mk	
0.5	< x < 6	+0.1	-
6	< x < 30	+0.2	-
30	< x < 120	+0.3	-
120	< x < 400	+0.5	-

Mould Denomination
 HIGH FLOW CHECK VALVE TUBE

Member Denomination
 HIGH FLOW CHECK VALVE TUBE_A-22

Date
 22/01/2022

Drawn by
 M. Cassani

N°Per.
 2

Moulding material
 See Table

Approved by
 Direzione C.A.

Dimensions

C.tion
 0.6%

Machine

Material

Mould weight

Eyebolts

Treatment

Mould dimensions

Scale

N° Cavities
 4

Order
 M1999


Customer code
 4010022

Customer part
 ST341A22x


A4

Sheet
 02

Date	Signature	Changes
		3
		2
		1

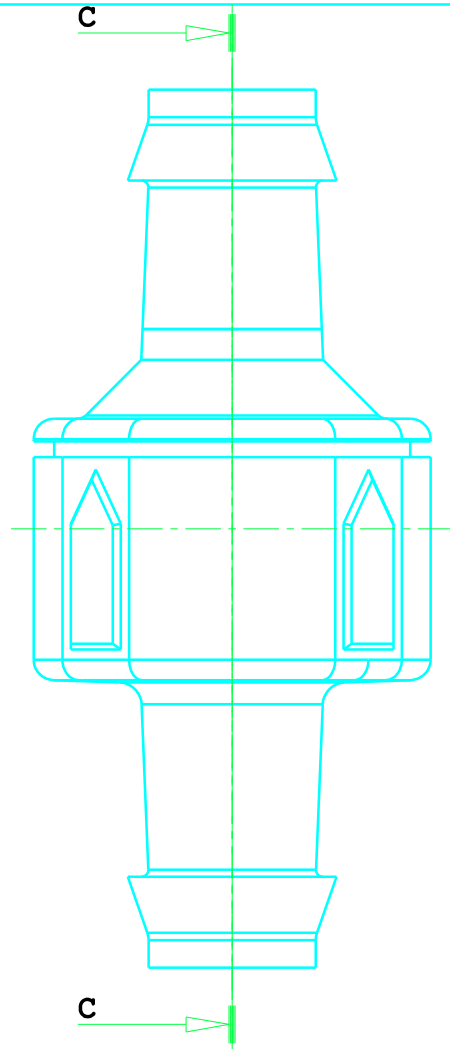

DRAWING DIMENSIONS REFER TO CYROLITE MATERIAL

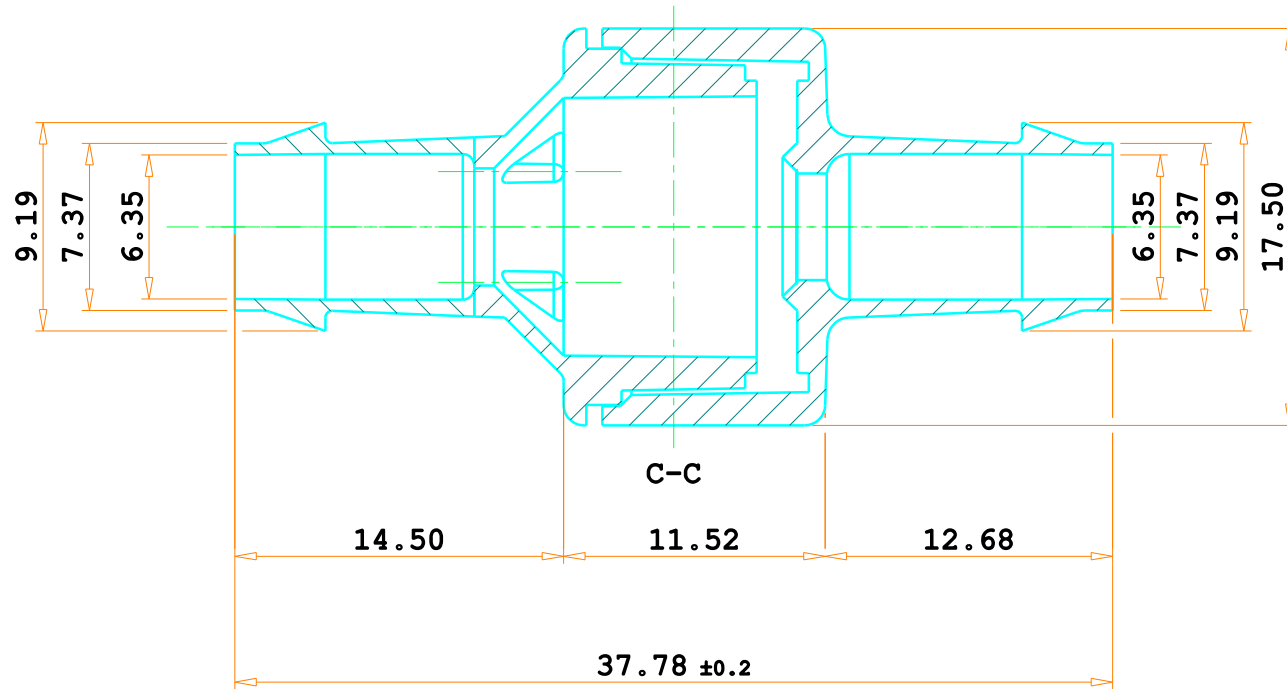

Product code	Moulding material
ST341A24C	CYROLITE
ST341A24T	TRIEX
ST341A24Z	ZYLAR



CARRO LUIGI Srl
 25030 RUDIANO (BS)
 Via Lavoro & Industria

M0803.04 Rev.00

General tolerances		ISO 2768-mk
0.5	< x < 6	+0.1
6	< x < 30	+0.2
30	< x < 120	+0.3
120	< x < 400	+0.5

Mould Denomination

HIGH FLOW CHECK VALVE TUBE

N° Cavities

4

Order

M1999

Member Denomination

HIGH FLOW CHECK VALVE TUBE_A-24

Customer code

4010022

Date
21/09/2022

Moulding material
See Table

C.tion
0.6%

Machine

Mould weight

Customer part
ST341A24 x

Drawn by

M. Cassani

Approved by

Direzione C.A.

Eyebolts

Mould dimensions

N°Par.

2

Dimensions

Material

Treatment

Scale

A4

Sheet 03



3 2 1

Changes

Signature

Date

DRAWING DIMENSIONS REFER TO CYROLITE MATERIAL

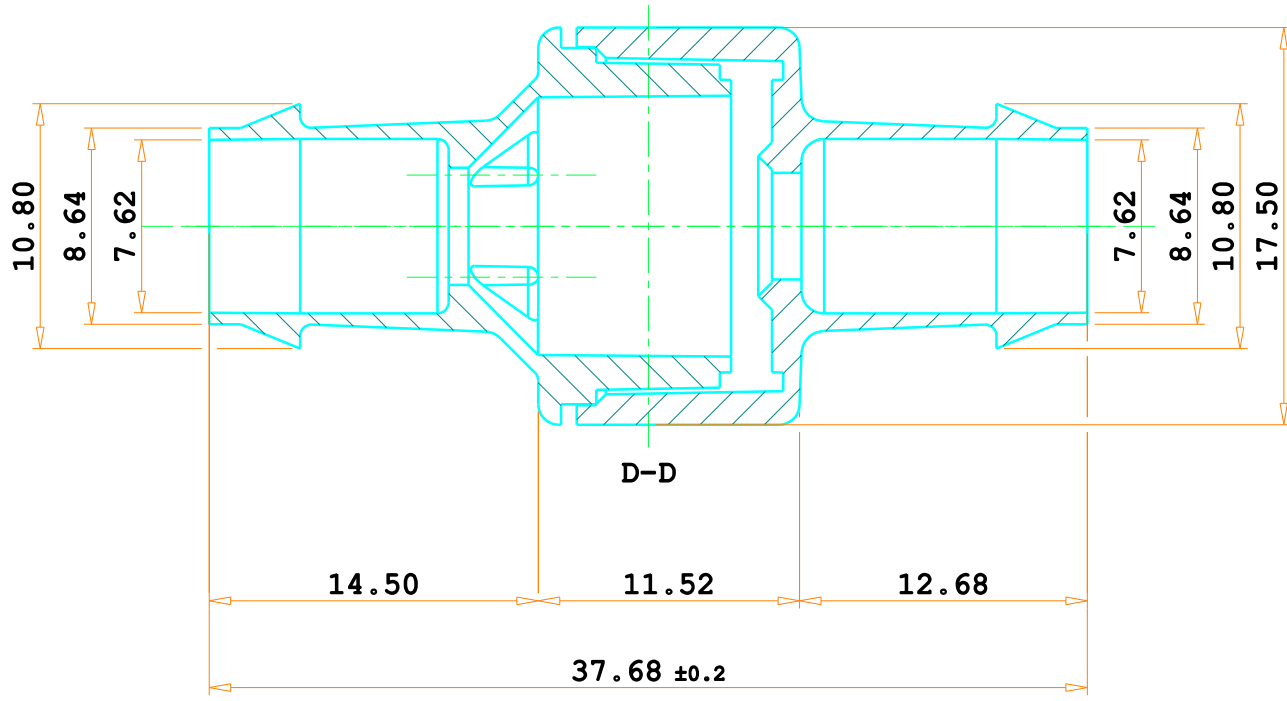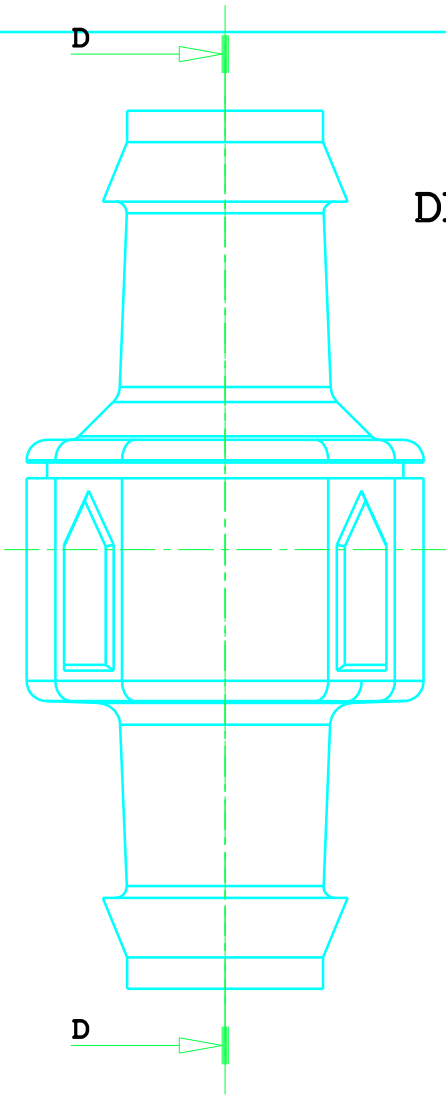

Product code	Moulding material
ST341A25C	CYROLITE
ST341A25T	TRIREX
ST341A25Z	ZYLAR

CARRO LUIGI Srl
 25030 RUDIANO (BS)
 Via Lavoro & Industria

M0803.04 Rev.00

General tolerances ISO 2768-mk

0.5 < x < 6	+0.1	.
6 < x < 30	+0.2	.
30 < x < 120	+0.3	.
120 < x < 400	+0.5	.

Mould Denomination
 HIGH FLOW CHECK VALVE TUBE

N° Cavities
 4

Order
 M1999

Member Denomination
 HIGH FLOW CHECK VALVE TUBE_A-25


Customer code
 4010022

Date 21/09/2022	Molding material See Table	C.tion 0.6%	Machine	Mould weight
Drawn by M. Cassani	Approved by Direzioe C.A.		Eyebolts	Mould dimensions
N°Par.	N°Pez. 2	Dimensions	Material	Treatment
				Scale

Customer part
 ST341A25x

Sheet
 04

3	2	1	Changes
			Signature
			Date

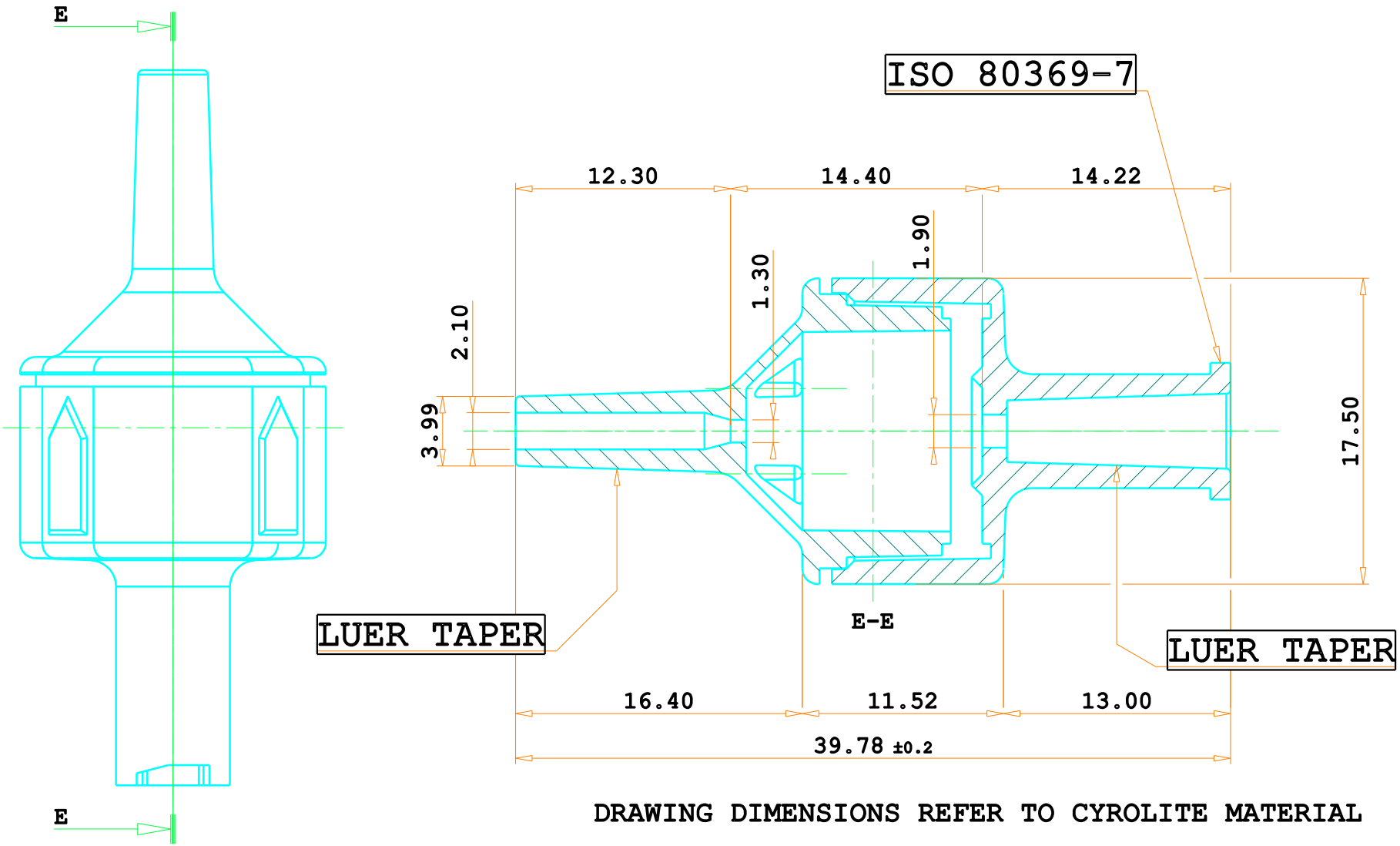

LUER TAPER

LUER TAPER

DRAWING DIMENSIONS REFER TO CYROLITE MATERIAL

Product code	Moulding material
ST341A31C	CYROLITE
ST341A31T	TRIREX
ST341A31Z	ZYLAR

CARRO LUIGI Srl
 25030 RUDIANO (BS)
 Via Lavoro & Industria

M0803.04 Rev.00

General tolerances	ISO 2768-mk
0.5 < x < 6	+0.1
6 < x < 30	+0.2
30 < x < 120	+0.3
120 < x < 400	+0.5

Mould Denomination
 HIGH FLOW CHECK VALVE TUBE

N° Cavities
 4

Order
 M1999

Member Denomination
 HIGH FLOW CHECK VALVE TUBE_A-31

Customer code
 4010022

Date	Moulding material	C.tion	Machine	Mould weight
21/09/2022	See Table	0.6%		
Drawn by	Approved by		Eyebolts	Mould dimensions
M. Cassani	Direzione C.A.			
N°Par.	N°Pez.	Material	Treatment	Scale
	2			

Customer part
 ST341A31x

Sheet
 05

3

2

1


Changes

Signature

Date

DRAWING DIMENSIONS REFER TO CYROLITE MATERIAL

LUER TAPER

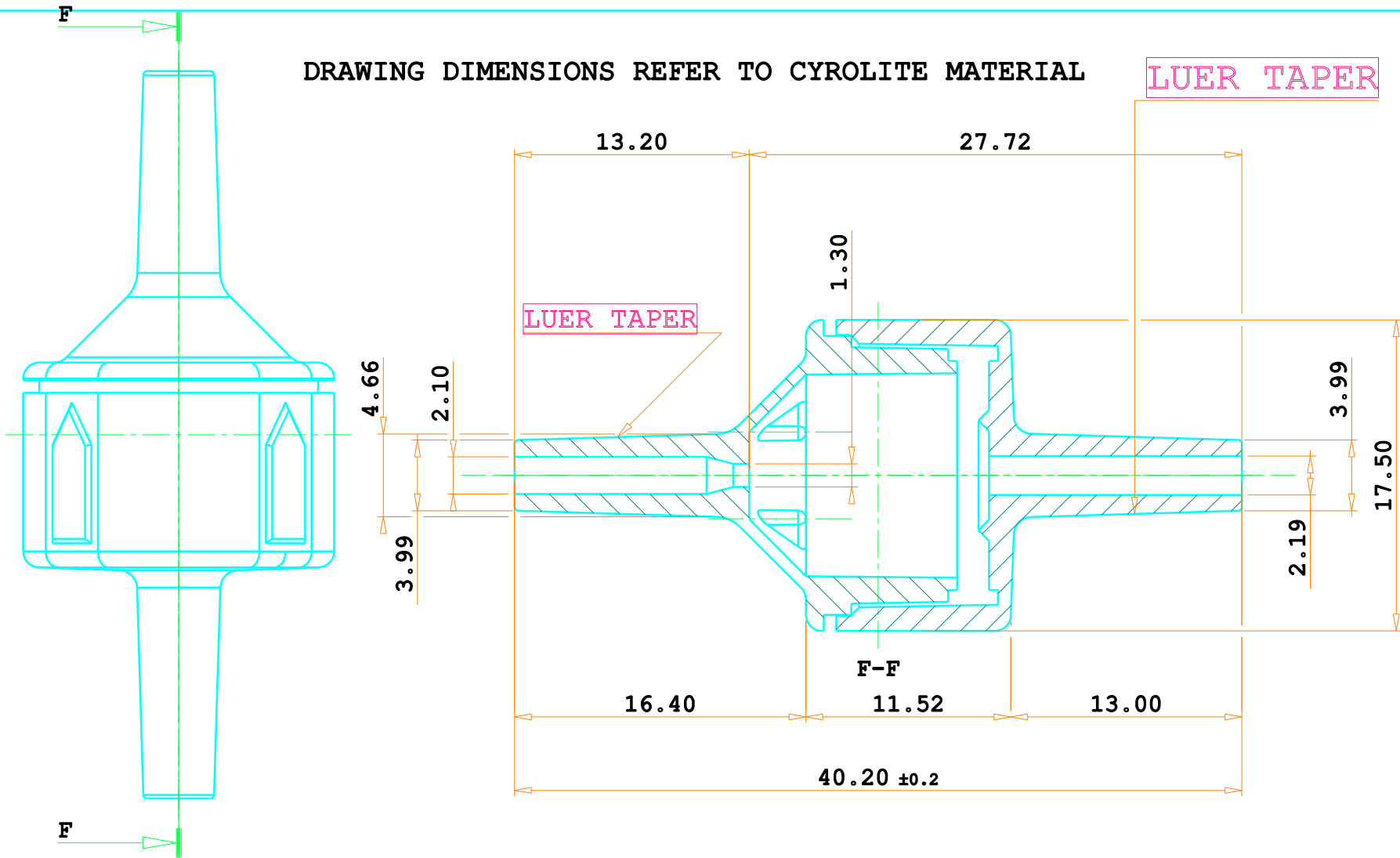

Product code	Moulding material
ST341A32C	CYROLITE
ST341A32T	TRIRES
ST341A32Z	ZYLAR

 **CARRO LUIGI Srl**
25030 RUDIANO (BS)
Via Lavoro & Industria

M0803.04 Rev.00

General tolerances	ISO 2768-mk
0.5 < x < 6	+0.1
6 < x < 30	+0.2
30 < x < 120	+0.3
120 < x < 400	+0.5

Mould Denomination
HIGH FLOW CHECK VALVE TUBE

N° Cavities
4

Order
M1999

Member Denomination
HIGH FLOW CHECK VALVE TUBE_A-32

Customer code
4010022

Date
21/09/2022

Moulding material
See Table

C.tion


Machine

Mould weight

Customer part
ST341A32x

Drawn by
M. Cassani

Approved by
Direzione C.A.

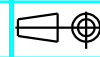

Eyebolts

Mould dimensions

N° Par.

N° Poz.
2

Dimensions

Material

Treatment

Scale

A4

Sheet 06

3


2

1


Changes

Signature


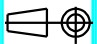

Date

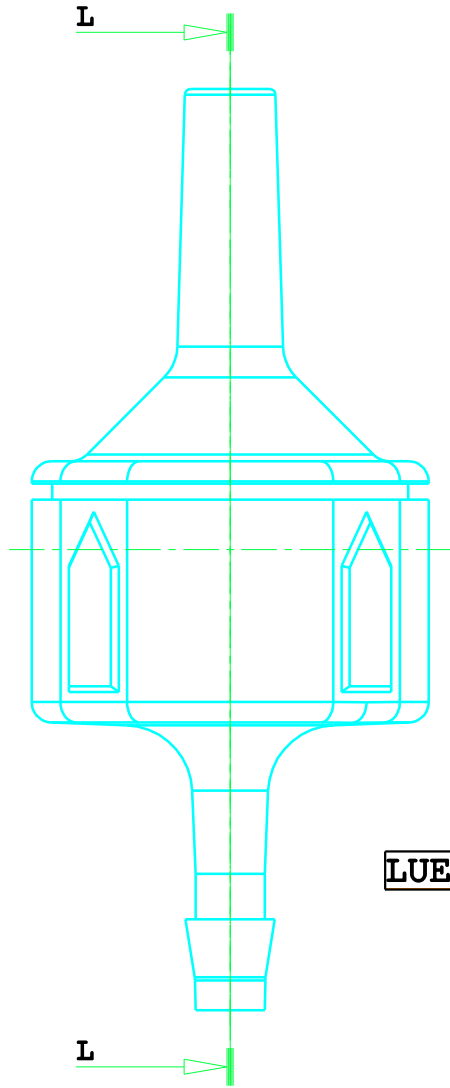



LUER TAPER

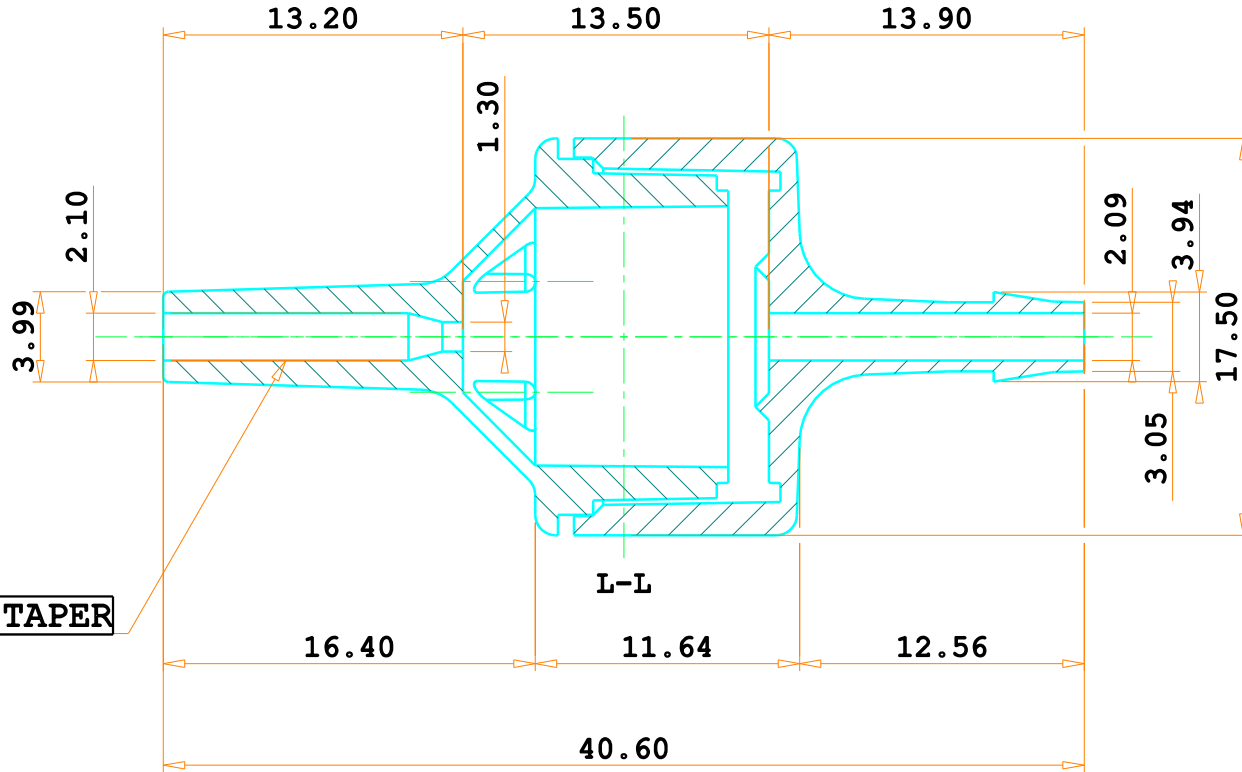

3	2	1	Changes
			Signature
			Date

 CARRO LUIGI Srl 25030 RUDIANO (BS) Via Lavoro & Industria	Mould Denomination HIGH FLOW CHECK VALVE TUBE			N° Cavities 4	Order M1999	
	Member Denomination HIGH FLOW CHECK VALVE TUBE_A21-A31 MIX			Customer code 4010022		
M0803.04 Rev.00	Date 21/09/2022	Molding material See Table	C.tion 0.6%	Machine ---	Mould weight ---	
General tolerances ISO 2768-mk 0.5 <x < 6 +0.1 . 6 <x < 30 +0.2 . 30 <x < 120 +0.3 . 120 <x < 400 +0.5 .	Drawn by M. Cassani	Approved by Direzione C.A.		Eyebolts ---	Mould dimensions ---	
	N° Par. 1+1	N° Poz. ---	Material ---	Treatment ---	Scale Scala 3:1	
						Sheet 01

DRAWING DIMENSIONS REFER TO CYROLITE MATERIAL

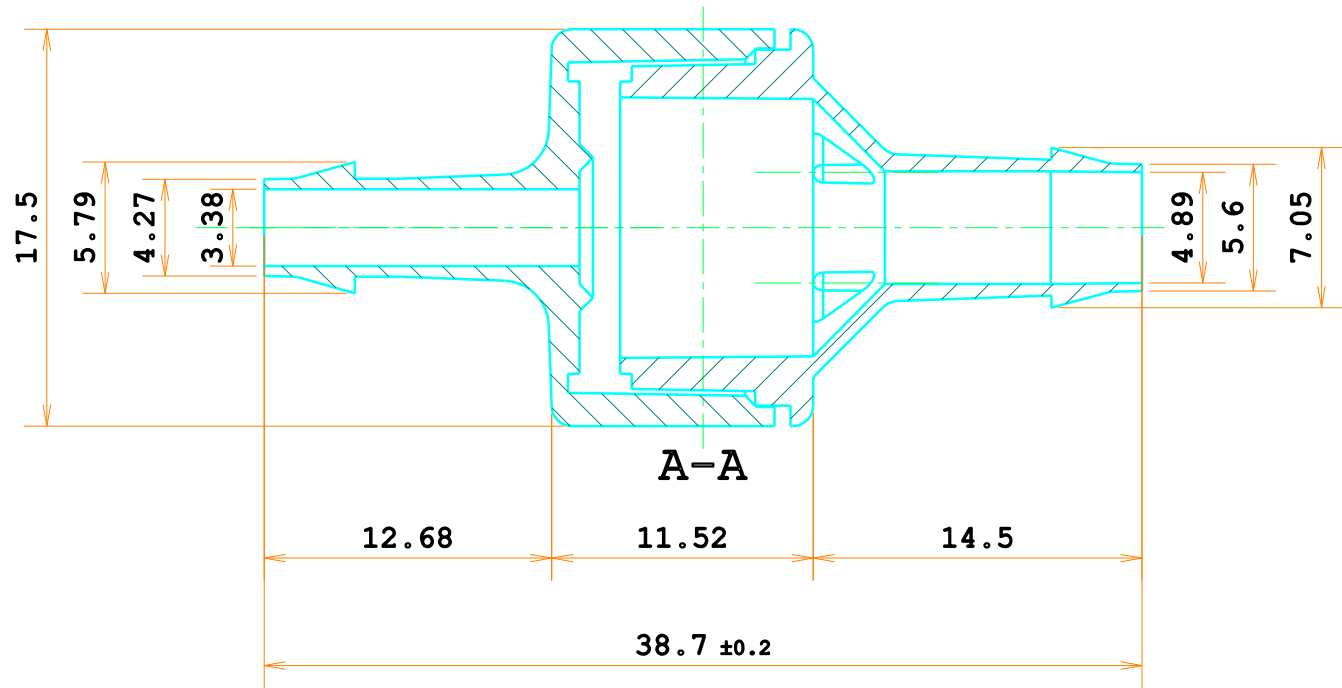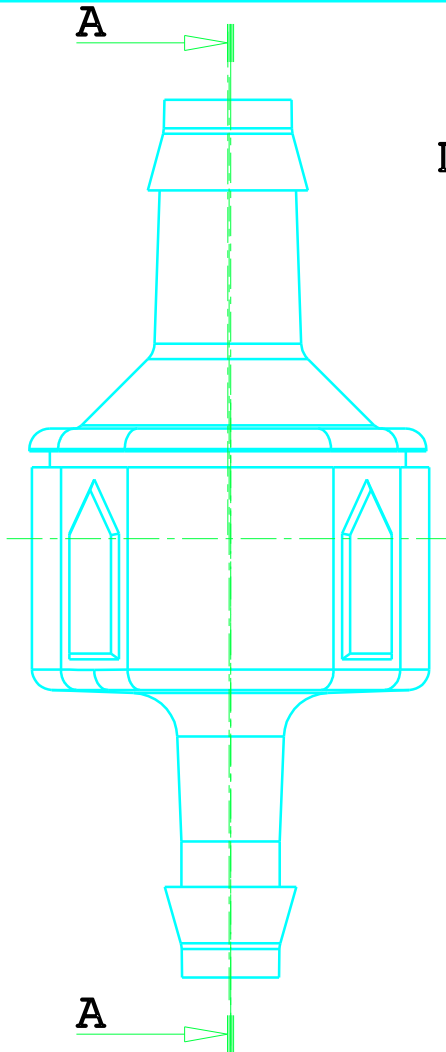


3
2
1

Changes

Signature

Date

PRODUCT CODE	MOULDING MATERIAL
ST341A22-B14Z	ZYLAR 550
ST341A22-B14T	PC TRIREX 3022
ST341A22-B14C	CYROLITE G20

 **CARRO LUIGI Srl**
25030 RUDIANO (BS)
Via Lavoro & Industria

M0803.04 rev.00

General Tolerances *ISO 2768-mk*

0.5 < x < 6	+0.1	.
6 < x < 30	+0.2	.
30 < x < 120	+0.3	.
120 < x < 400	+0.5	.

Mould Denomination
HIGH FLOW CHECK VALVE

N° Cavities
//

Order
//

Member Denomination

HIGH FLOW CHECK VALVE A-22 TO BARBED 1/4 (A-23)

Customer code
//

Date
20/02/23

Molding Material
SEE TABLE

C.tion
0.6%

Machine
//

Mould Weight
//

Customer part
ST341A22-B14x

Drawn By
Cassani M.

Approved By
Direzione C.A.



Eyebolts
//

Mould Dimensions
//

N°Par.
//

N°Pez.
1+1

Dimensions
//

Material
//

Treatment
//

Scale
Scale 3:1

A4

Sheet
-/-